

WARNING!!!

When operating with clamp:

- 1) open the clamp for necessary width outside the mouth before placing it on a tooth;
- 2) avoid high-speed opening;
- 3) open the clamp for width not more than 1,5 mm exceeding tooth width (undue force may cause clamp premature failure because of metal fatigue);
- 4) fix the clamp with floss to avoid clamp swallowing.

Cleaning and sterilization instructions:

- 1) Submerge instruments in cleaning agent solution with neutral pH for 20 min;
- 2) Rinse the instruments thoroughly with purified water for 3-5 min;
- 3) Perform standard moist heat / steam sterilization procedure following the sterilizer instructions.

WARNING!!!

- 1) Do not use cleaning agents with high percentage of chlorine and cleaners containing oxalic acid;
- 2) Do not keep instruments in liquid medium for more than 3 hours.